

Please follow these instructions in order to remove and install the conductive treatment electrodes integrated into ear cushions of your Cervella headset. The ear cushions should be replaced when they become worn out or excessively dirty. You can purchase replacement ear cushions at [www.cervella.us](http://www.cervella.us).

**NOTE: DISCONNECT THE CONNECTING CABLE FROM HEADSET BEFORE CONTINUING**

### STEP 1 – REMOVING EAR CUSHIONS

To remove an ear cushion, grab the ear cushion firmly, twist it **COUNTERCLOCKWISE** and lift the ear cushion away.



## STEP 2 – INSTALLING EAR CUSHIONS

Locate the LEFT ear cushion and LEFT earpiece of the Cervella headset as marked on the headband. Position the ear cushion slightly to the left of vertical (at a 10 o'clock position), press firmly to engage the four (4) tabs (circled) and turn **CLOCKWISE** until the L letter is vertical. Verify that all four tabs are fully engaged into the ear cushion. Repeat for the Right ear cushion.

**Note:** If the four tabs are not fully engaged (indicated by a gap between the ear cushion and the headset), remove the ear cushion and try again.



## STEP 3 – VERIFY CERVELLA OPERATION

Connect the headset to the Cervella main device and start the stimulation. The LED on the main device should glow solid **BLUE**. Next, press and hold the gold portions of the left and right ear cushions together. The LED on the device should now be blinking **BLUE** indicating proper installation and functionality.

**Note:** If the Cervella LED remains solid blue and does not blink when the ear cushions are touching, remove each ear cushion and gently bend each gold-plated connector (circled in **ORANGE**) UP under each ear cushion, re-assemble, and re-test.

